

PCI Drug Screening Testing Information

Precision Oral Scan

The Precision Oral Scan drug testing kit is an oral collection swab that takes about 10 minutes to result.

With this new test and Covid guidelines comes new testing procedures, below is some important information regarding the new testing procedures.

1. Upon arrival participants will have to sit supervised by appropriate staff for 15 minutes before being tested.
2. Participants should avoid placing anything (food, drink, gum or tobacco) in their mouth for at least 10 minutes prior to testing.
3. Participants will have to wait at the testing facility until the test has been completed.
(Any participant with a positive oral test will automatically have to submit a urine specimen to be sent out for additional testing)
4. Program staff will have 2 testing days available, it is the participants/ participant's parent/guardians responsibility to make sure that arrangements are made to be tested on the 2 testing days made available. Failure to be tested within the provided time frame will result in the participant becoming ineligible.

Testing Instructions

Using the provided swab, sweep the inside of the mouth (cheeks, gum and tongue) several times, and then hold the swab in your mouth until the saturation indicator changes color.

DO NOT bite, suck, or chew on the sponge!

Testing Procedure

1. The participant will enter the testing facility alone (Parents must wait outside). Upon entering the testing facility the participant's mouth will be inspected to make sure there is nothing in their mouth (food, drink, gum, etc...).
2. Compliance with PCI Covid guidelines must be followed. The participant must wear a mask to enter the testing facility. Waiting area for the participants will be marked with appropriate social distancing guidelines.
3. Once the 15 minute wait time has been met, the participant will be given instructions on how to swab their mouths with the test to collect their own specimen. The testing administrator will observe and make sure that the participants are following the instructions for collection.
4. Once the specimen has been collected the testing administrator will take up all test and go to an approved secure location to result test.
5. Each participant will be called individually to receive the results of their swab. Participants will be given a slip of paper which indicates if the test was positive or negative. Positive results will be called last to maintain confidentiality.
6. Any positive results will automatically have to collect a urine specimen to confirm preliminary results.